Medivical

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No. 18, Kar Afarinan St. 14/1
Isfahan Great Industrial Town
Isfahan, Iran
Postal Code: 83311-34319

☎ +98 31 45647282-92
☎ +98 31 45647291
🌐 www.medivical.com
✉ info@medivical.com

Graphic Design **Rohamotamedi**
Photography **Makanshafiei**

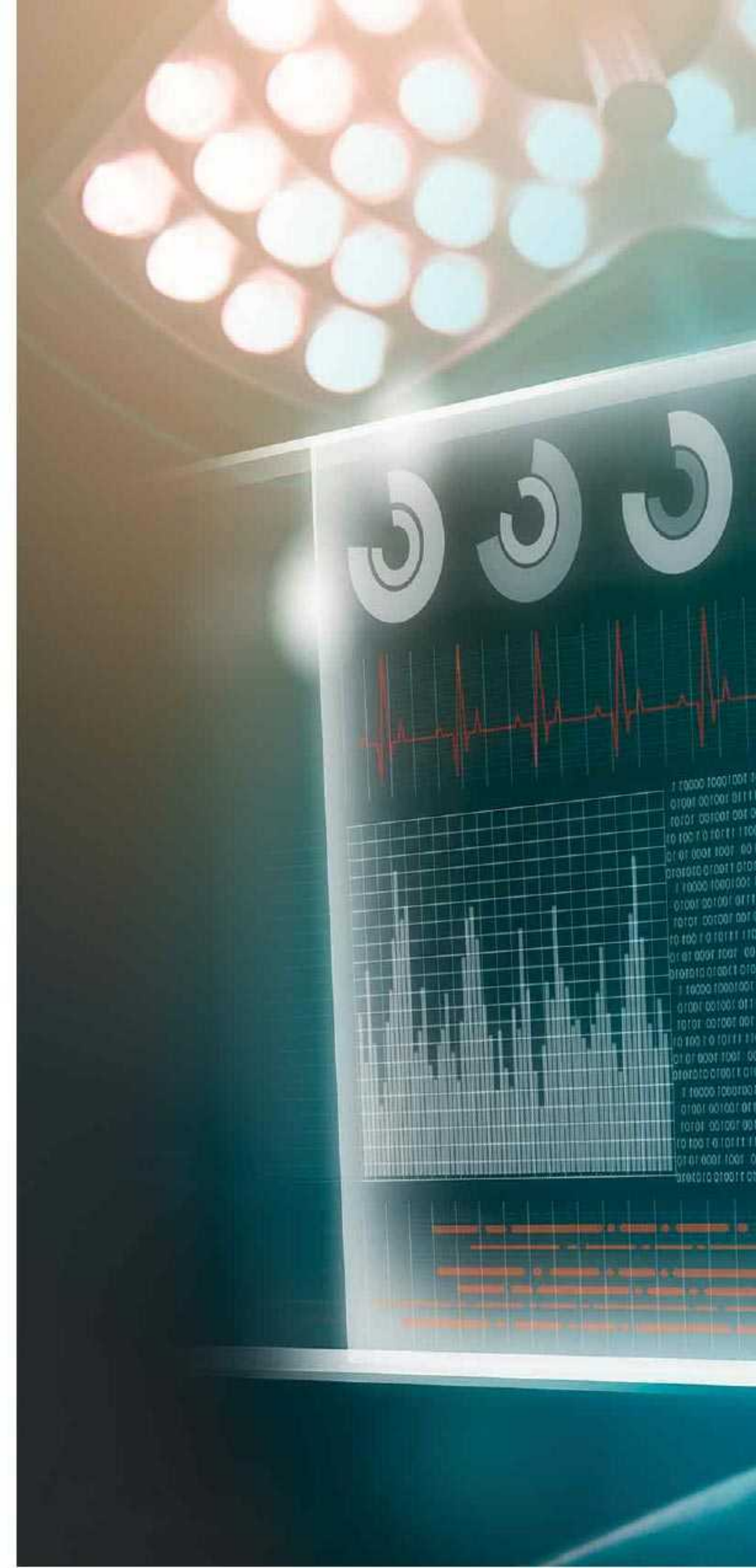
Medivical

VIRA TREATA MODERN CO.

No. 18, Kar Afarinan St. 14/1
Isfahan Great Industrial Town
Isfahan, Iran
Postal Code: 83311-34319

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Application

- General Surgery
- Liposuction
- Endoscopy
- Emergency Unit
- Air Way Suction
- Neuro Surgery
- Wound Drainage
- Plastic Surgery

Fast Facts

- Reliability meets modern technology
- One-touch flow start
- Two years warranty
- ISO 13485
- High-quality, durable components
- Hygienic through intelligent construction
- In accordance with international standards

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- 1 Vacuum Gauge in kPa and mmHg
- 2 Safety Set
- 3 Membrane Vacuum Regulator
- 4 On/Off Touch Button - Standby Indicator
- 5 Suction Tube Holder
- 6 Standard Rail
- 7 Change-Over Valve
- 8 5L or 3L Reusable Collection System
- 9 Four Castors with Brakes

Technical Highlights

- High Vacuum: Up to -90 kPa , -675 mmHg
- High Flow: Up to 60 L/min
- Dimension: 59 × 49 × 100 cm
- Weight: 18 Kg
- Protection Against Water: IP×1
- Fast Vacuum Build Up
- Very Quiet in Operation and Low-noise System
- Very Hygienic and Easy to use as well as versatile
- Membrane Vacuum regulator
- Vacuum gauge in kPa and mmHg
- Carrying handle
- Four Castors with brakes
- Smart sensor for over fill protection of Canister
- Mechanical over fill protection
- Touch ON/OFF button
- Standby indicator
- 5L and 3L reusable collection system
- 5.5L and 3.5 L disposable collection system
- Change-over valve
- Safety set

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Easy to Use and Virtually Indestructable

The Medivical Reusable Collection System consists of autoclavable suction canisters and lids with built-in overflow protection.

All two jar sizes [3L and 5L] have the same lid size to make handling easy and reduce logistical considerations.

The system is meant for flexible use: choose between the complete system or specific components and accessories; Medivical can help you select the products that are right for you.

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Benefits of Reusable Collection System

Medivicals Reusable Collection System consists of multiple components and accessories. It is easy to use, easy to set up and hygienic. It provides simple, fast operation, is long-lasting and safe by design.

- Tubing can be quickly connected canisters to the system distinct vacuum and patient ports.
- The system comes with built-in mechanical overflow protection and easy-to-read graduated markings to indicate fluid volumes.
- The quick-to-connect components make the system ready to use in just seconds.

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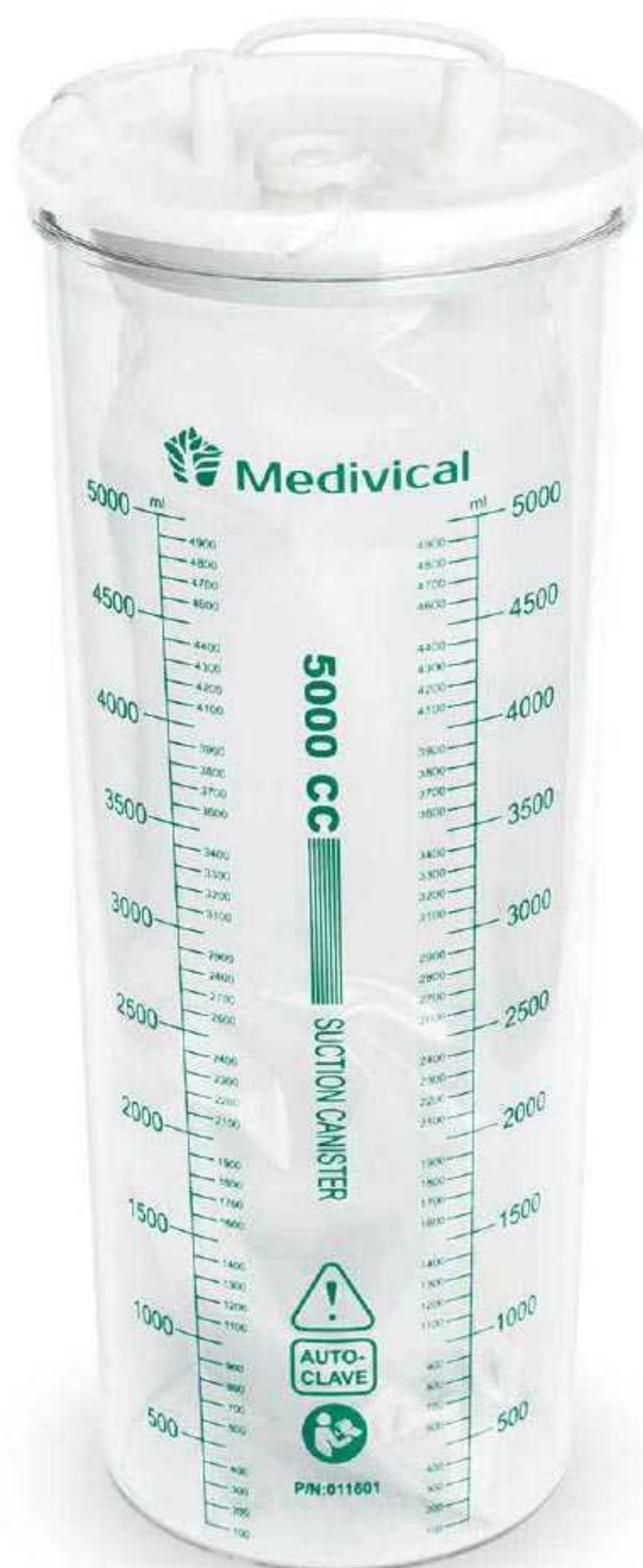
Easy to Use

The Vira Treata Modern Disposable Collection System offers a whole range of advantages that makes it easy to implement in the hospital and clinic. Simply place the one-piece liner into the jar and it is ready to use in seconds. As the vacuum is located direct to the jar the liner can be conveniently removed and replaced without removing the vacuum tube. A one handed liner removal easy. If large amounts of fluid need to be suctioned, the system can be used in serial or with the change-over valve.

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Hygienic

The system offers an overflow filter, integrated in each liner with fluid navigation to prevent back-flow and reduce foaming/back splashing. The material of the liner consists of layer for durability and odour control. A special feature is the pouch that keeps the solidifier contained until the liner is in use. This means no spillage or inconvenience for the staff.

Fast Facts

- Easy to set up and ready to use in seconds.
- Liner can be removed from jar independently from vacuum tubing.
- Overflow protection filter in each liner.
- The fluid navigation reduces foaming.
- Serial connection to allow suctioning of large volumes of fluid.
- Solidifier in pouch keeps solidifier contained until liner is in use.
- 3500cc and 5500cc for easy storage and logistics.

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